
THUASNE

Bandage rotulien

fr	Bandage rotulien.....	7
en	Patellar bandage.....	8
de	Patella-Bandage.....	10
nl	Knieschijfverband.....	12
it	Bendaggio rotuleo.....	14
es	Venda rotuliana.....	16
pt	Ligadura rotuliana.....	18
da	Knæskals-bandage.....	19
fi	Polvilumpioside.....	21
sv	Knäskålsförband.....	23
el	Επίδεσμος επιγονατίδας.....	25
cs	Obvaz na čéšku.....	27
pl	Bandaż do owijania rzepki.....	28
lv	Ceļa locītavas saite.....	30
lt	Girnelės juosta.....	32
et	Põlvekedra side.....	34
sl	Patelarni trak.....	35
sk	Bandáž na patelárnu šľachuPopis/Použitie.....	37
hu	Patellapánt.....	39
bg	Пателарна лента.....	41
ro	Bandaj rotulian.....	43
ru	Поддерживающая повязка для коленного сустава.....	45
hr	Patelarni zavoj.....	47
zh	髌骨绷带.....	48
ar	لفافة رصفية.....	51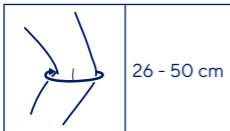

fr	Circonférence du genou
en	Knee circumference
de	Knieumfang
nl	Omvang van de knie
it	Circonferenza del ginocchio
es	Circunferencia de la rodilla
pt	Circunferência do joelho
da	Omkreds af knæ
fi	Polven ympärys
sv	Omkrets runt knä
el	Περίμετρος του γόνατος
cs	Obvod kolena
pl	Obwód kolana
lv	Celgala apkārtmērs
lt	Kelio apimtis
et	Põlve ümbermõõt
sl	Obseg kolena
sk	Obvod kolena
hu	Térd körfogata
bg	Обиколка на коляното
ro	Circumferința genunchiului
ru	Диаметр окружности колена
hr	Opseg koljena
zh	膝关节周长
ar	محيط الركبة

fr	Pression localisée	Effet proprioceptif
en	Targeted pressure	Proprioceptive effect
de	Lokaler Druck	Propriozeptive Wirkung
nl	Lokale druk	Proprioceptieve werking
it	Pressione localizzata	Effetto proprioceptivo
es	Presión localizada	Efecto propioceptivo
pt	Pressão localizada	Efeito proprioceptivo
da	Lokalt tryk	Proprioceptisk effekt
fi	Paikallinen paine	Liikeaistiin kohdistuva vaikutus
sv	Lokaliserat tryck	Proprioceptiv effekt
el	Εντοπισμένη πίεση	Αποτέλεσμα ιδιοδεκτικότητας
cs	Lokalizovaný tlak	Proprioceptivní účinek
pl	Ucisk miejscowy	Propriocepcja
lv	Lokalizēts spiediens	Proprioceptīvs efekts
lt	Konkrečios vietos suspaudimas	Propriocepčinis poveikis
et	Paikne surve	Proprioceptiivne toime
sl	Lokaliziran pritisk	Proprioceptivni učinek
sk	Presne zacielený tlak	Proprioceptívny účinok
hu	Lokalizált nyomás	Proprioceptív hatás
bg	Локализирано налягане	Проприоцептивен ефект
ro	Presiune localizată	Efect proprioceptiv
ru	Локализованное давление	Проприоцептивный эффект
hr	Lokalizirani pritisak	Učinak proprioceptivnog vježbanja
zh	局部施压	本体感
ar	ضغط موضعي	مفعول استقبال الحس العميق

MD

fr	Dispositif médical	Un seul patient - à usage multiple
en	Medical Device	Single patient - multiple use
de	Medizinprodukt	Einzelner Patient - mehrfach anwendbar
nl	Medisch hulpmiddel	Eén patiënt - meervoudig gebruik
it	Dispositivo medico	Singolo paziente - uso multiplo
es	Producto sanitario	Un solo paciente - uso múltiple
pt	Dispositivo médico	Paciente único - várias utilizações
da	Medicinsk udstyr	Enkelt patient - flergangsbrug
fi	Lääkinnällinen laite	Potilaskohtainen - voidaan käyttää useita kertoja
sv	Medicinteknisk produkt	En patient - flera användningar
el	ιατροτεχνολογικό προϊόν	Πολλαπλή χρήση - σε έναν μόνο ασθενή
cs	Zdravotnický prostředek	Jeden pacient - vícenásobné použití
pl	Wyrób medyczny	Wielokrotnie użycie u jednego pacjenta
lv	Medicīniska ierīce	Viens pacients - vairākkārtēja lietošana
lt	Medicinos priemonė	Vienas pacientas - daugkartinis naudojimas
et	Meditsiiniseade	Ühel patsiendil korduvalt kasutatav
sl	Medicinski pripomoček	En bolnik - večkratna uporaba
sk	Zdravotnícka pomôcka	Jeden pacient - viacnásobné použitie
hu	Orvostechikai eszköz	Egyetlen beteg esetében többször újrahasználható
bg	Медицинско изделие	Един пациент - многократна употреба
ro	Dispozitiv medical	Un singur pacient - utilizare multiplă
ru	Медицинское изделие	Многократное использование для одного пациента
hr	Medicinski proizvod	Jedan pacijent - višestruka uporaba
zh	医疗器械	一位患者-多次使用
ar		لمريض واحد - متكرر الاستعمال جهاز طبي

Lire attentivement la notice

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PATELLAR BANDAGE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the product's sizing.

One size. Bilateral product.

The device is composed of:

- a main fabric ❶ linked to a strap ❷ allowing localized contention and maintain,
- a silicone insert ❸ located on the patellar tendon.

Composition

Textile components: polyamide - polyurethane - polyethylene - cotton.

Rigid components: polyamide - polyoxymethylene - silicone.

Properties/Mode of action

The patellar bandage applies local pressure to the patellar tendon and supports the patella, thereby absorbing vibrations.

Proprioceptive effect thanks to the combination of silicone with dimples, which creates a massage effect and a fabric band, which provides contention.

Indications

Patella joint or ligament inflammation and tendinopathy (runner's knee and jumper's knee syndrome).

Mild patellar instability.

Anterior knee or patella pain.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Ensure that the knee circumference is compatible with the product circumference.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

This product is intended for the treatment of a given condition. Its duration of use is limited to this treatment only.

For hygiene and performance reasons, do not re-use the product for another patient.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Before any sports activity, check the compatibility of the use of this medical device with your healthcare professional.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

- First unfasten the tightening strap.
 - The patellar bandage must be placed over the patellar tendon.
- The patellar tendon must be positioned at the center of the silicone insert.
- Next, re-fasten the strap around the knee.
 - Make sure that the Thuasne logo is on the upper part of the bandage.

Care

Hand wash. Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2026701 (2021-04)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186 – capital 1 950 000 euros