

THUASNE®

en	Sport patellar bandage	4
fr	Bandage rotulien sport	5
de	Sport-Patellabandage	7
nl	Knieschijfverband sport	9
it	Bendaggio rotuleo per uso sportivo	11
es	Venda rotuliana para uso deportivo	13
pt	Ligadura rotuliana para uso desportivo	14
da	Knæskalsbind sport	16
fi	Polvilumpioside urheilukäyttöön	18
sv	Knäskålsförband sport	19
el	Αθλητικός επίδεσμος επιγονατίδας	21
cs	Češková ortéza sport	23
pl	Bandaż do owijania rzepki sportowy	24
lv	Sporta ceļa skriemeļa apsējs	26
lt	Girnelės juosta sportui	28
et	Spordi põlvekedra side	29
sl	Trak za pogačico šport	31
sk	Športová patelárna bandáž	33
hu	Térdkalács szalag sportoláshoz	34
bg	Пателарен бандаж за спорт	36
ro	Bandaj rotulian sport	38
hr	Sport patelarna traka	39

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

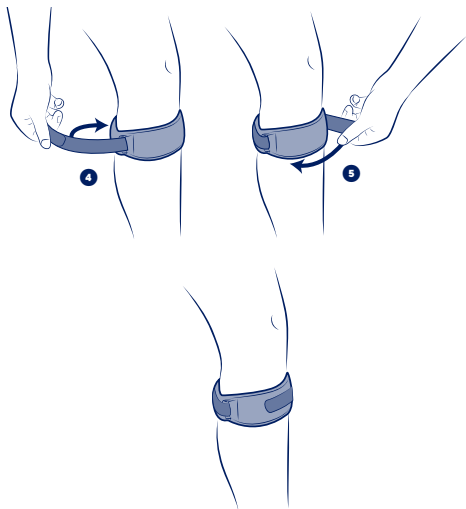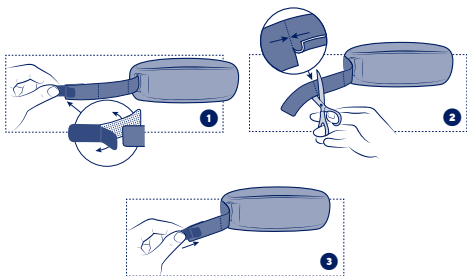en	Universal size. Knee circumference
fr	Taille unique. Circonférence du genou
de	Einheitsgröße. Knieumfang
nl	Eén maat. Omvang van de knie
it	Taglia unica. Circonferenza del ginocchio
es	Talla única. Circunferencia de la rodilla
pt	Tamanho único. Circunferência do joelho
da	Kun én størrelse. Omkreds af knæ
fi	Yksi koko. Polven ympäry
sv	En storlek. Omkrets runt knä
el	Ένα μέγεθος. Περίμετρος του γόνατος
cs	Univerzální velikost. Obvod kolena
pl	Rozmiar uniwersalny. Obwód kolana
lv	Universāls izmērs. Ceļgala apkārtmērs
lt	Vienas dydis. Kelio apimtis
et	Üks suurus. Põlve ümbermõõt
sl	Enotna velikost. Obseg kolena
sk	Univerzálna veľkosť. Obvod kolena
hu	Egy méret. Térd körfogata
bg	Един размер. Обиколка на коляното
ro	Mărimē universală. Circumferința genunchiului
hr	Univerzalna veličina. Opseg koljena

SPORT PATELLAR BANDAGE

Description/Destination

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the product's sizing.

Patellar compression bandage.

The device is composed of:

- a textile strap combined with a strapping strap providing targeted compression and support,
- a silicone insert positioned over the patellar tendon,
- an adjustable touch fastening system to adjust the strapping strap.

Composition

Textile components: polyamide - polyurethane - polyethylene - elastane - synthetic rubber (SBR) - polyester.

Rigid components: polyamide - polyoxymethylene - silicone.

Properties/Mode of action

The patellar bandage applies local pressure to the patellar tendon and supports the patella, thereby absorbing vibrations.

Proprioceptive effect thanks to the combination of silicone with dimples, which creates a massage effect and a fabric band, which provides contention.

Indications

Joint or ligament inflammation of the patella, patellar tendinitis (jumper's knee) or femoropatellar syndrome (runner's knee).

Mild patellar instability.

Anterior knee or patella pain.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Ensure that the knee circumference is compatible with the product circumference.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

Consult a healthcare professional in case of uncertain diagnosis.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

In case of history of venous or lymphatic disorders, consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

It is recommended to wear the device directly on the skin, unless contraindicated.

- First unfasten the tightening strap.
- The patellar bandage must be placed over the patellar tendon.

The patellar tendon must be positioned at the center of the silicone insert.

To adjust the patellar bandage, a removable touch fastening and tightening strap pre-cutting system is available:

- Remove the removable touch fastener ❶.
- Cut the tightening strap at the marker corresponding to your size ❷.
- Reinstall the removable touch fastener ❸.
- Pass the tightening strap back through the buckle then install the patellar bandage ❹.
- Next, re-fasten the strap around the knee ❺.
- Make sure that the Thusne logo is on the upper part of the bandage.

Care/Maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Do not machine wash the device more than 30 times. Machine wash at 30°C (delicate programme). Close the self-fastening tabs before washing. If possible use a washing net. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2048901 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186