
**THUASNE®**

# Ligaflex® Manu

<b>fr</b>	Attelle d'immobilisation poignet-pouce.....	4
<b>en</b>	Wrist and thumb immobilisation splint .....	5
<b>de</b>	Orthese zur Ruhigstellung von Handgelenk und Daumen .....	6
<b>nl</b>	Immobilisatiebrace voor pols en duim.....	8
<b>it</b>	Tutore d'immobilizzazione del polso - pollice.....	9
<b>es</b>	Férula de inmovilización de muñeca - pulgar.....	11
<b>pt</b>	Tala de imobilização de punho-polegar .....	12
<b>da</b>	Skinne til immobilisering af håndled-tommelfinger.....	13
<b>fi</b>	Tukeva ranne-peukalolasta.....	15
<b>sv</b>	Ortos för immobilisering av handled/tumme .....	16
<b>el</b>	Νάρθηκας ακινητοποίησης καρπού-αντίχειρα.....	17
<b>cs</b>	Ortéza imobilizační zápěstí a palce.....	19
<b>pl</b>	Orteza nadgarstkowo-ręczno-palcowa .....	20
<b>lv</b>	Plaukstu locītavas-īkšķa imobilizācijas šina .....	21
<b>lt</b>	Riešio ir nykščio imobilizavimo įtvaras.....	23
<b>et</b>	Randme ja põidla fikseerimise lahas .....	24
<b>sl</b>	Imobilizacijska ploščica za zapestje - palec.....	25
<b>sk</b>	Imobilizačná dlaha na zápästie-palec.....	27
<b>hu</b>	Csukló-hüvelykujj rögzítő sín.....	28
<b>bg</b>	Шина за обездвижване на китка-палец.....	29
<b>ro</b>	Atelă de imobilizare a încheieturii mâinii și a degetului mare.....	31
<b>ru</b>	Шина для иммобилизации запястья - большого пальца.....	32
<b>hr</b>	Udlaga za imobilizaciju palčanog zgloba .....	34
<b>zh</b>	手腕和拇指固定夹板 .....	35
<b>ar</b>	جبيرة تثبيت الإبهام والمعصم.....	39

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال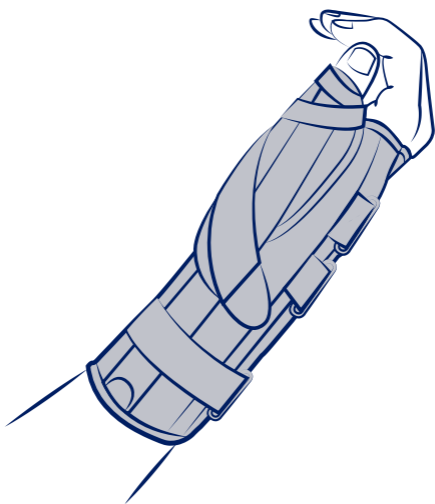

	0	10 - 13 cm
	1	13 - 15 cm
	2	15,5 - 17,5 cm
	3	18 - 20 cm
	4	20,5 - 23 cm

<b>fr</b>	Tour de poignet
<b>en</b>	Wrist circumference
<b>de</b>	Handgelenkumfang
<b>nl</b>	Polsomtrek
<b>it</b>	Circonfrenza del polso
<b>es</b>	Contorno de la muñeca
<b>pt</b>	Perímetro do punho
<b>da</b>	Håndleddets omkreds
<b>fi</b>	Ranteen ympärysmitta
<b>sv</b>	Handledens omkrets
<b>el</b>	Περίμετρος καρπού
<b>cs</b>	Obvod zápěstí
<b>pl</b>	Obwód nadgarstka
<b>lv</b>	Plaukstas locītavas apkārtmērs
<b>lt</b>	Riešo apimtis
<b>et</b>	Randme ümbermõõt
<b>sl</b>	Obseg zapestja
<b>sk</b>	Obvod zápästia
<b>hu</b>	Csukló kerülete
<b>bg</b>	Обиколка на китката
<b>ro</b>	Circumferința încheieturii
<b>ru</b>	Обхват запястья
<b>hr</b>	Obujam ručnog zgloba
<b>zh</b>	腕围
<b>ar</b>	محيط الرسغ

<b>fr</b>	Immobilisation
<b>en</b>	Immobilisation
<b>de</b>	Ruhigstellung
<b>nl</b>	Immobilisatie
<b>it</b>	Immobilizzazione
<b>es</b>	Inmovilización
<b>pt</b>	Imobilização
<b>da</b>	Støtte
<b>fi</b>	Immobilisoi
<b>sv</b>	Immobilisering
<b>el</b>	Ακινητοποίηση
<b>cs</b>	Znehybnění
<b>pl</b>	Unieruchomienie
<b>lv</b>	Imobilizācija
<b>lt</b>	Imobilizavimas
<b>et</b>	Liikumatuks muutmine
<b>sl</b>	Imobilizacija
<b>sk</b>	Imobilizácia
<b>hu</b>	Rögzítés
<b>bg</b>	Обездвижване
<b>ro</b>	Imobilizare
<b>ru</b>	Иммобилизация
<b>hr</b>	Imobilizacija
<b>zh</b>	固定
<b>ar</b>	التثبيت

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## **WRIST AND THUMB IMMOBILISATION SPLINT**

### **Description/Destination**

This device is a wrist-thumb splint providing joint immobilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition**

Textile components: polyamide - polyester - cotton - elastane.

Rigid components: polyoxymethylene - aluminum.

### **Properties/Mode of action**

Immobilisation of the wrist and thumb via 2 rigid, malleable and anatomical palmar stays.

Aerated, foam lined fabric.

Dual strap system ensuring even compression.

### **Indications**

Thumb sprain.

Wrist and/or thumb sprain.

Non surgical scaphoid fracture.

Non surgical wrist fracture.

Post-operative or post-traumatic wrist/thumb immobilisation.

Wrist and/or thumb tendinopathies.

De Quervain tenosynovitis.

Rheumatoid arthritis.

Lateral epicondylitis.

Conservative treatment of carpal tunnel syndrome.

Thumb/wrist osteoarthritis.

Neuro-muscular impairments (contracture, hand drop or spasticity...).

### **Contraindications**

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

## **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

It is recommended that an adult supervises the application and use of the product by a child.

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

## **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

## **Instructions for use/Application**

If necessary, shape the stays.

Open the splint and position the thumb and the wrist inside the product.

Adjust the tightening system (wrist then thumb) to close the splint.

## **Care/Maintenance**

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Remove stay before washing. Replace in the original location before next use. Close the self-fastening tabs before washing. Machine washable at 40 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

## **Storage**

Store at room temperature, preferably in the original packaging.

## **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2051001 (2023-12)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186