


**THUASNE®**

<b>en</b>	<b>Sport thumb strapping</b> .....	4
<b>fr</b>	<b>Strapping de pouce sport</b> .....	5
<b>de</b>	<b>Strapping Sport-Daumenbandage</b> .....	6
<b>nl</b>	<b>Duimbrace met elastische banden sport</b> .....	7
<b>it</b>	<b>Strapping per il pollice per uso sportivo</b> .....	9
<b>es</b>	<b>Strapping de pulgar para uso deportivo</b> .....	10
<b>pt</b>	<b>"Strapping" de polegar para uso desportivo</b> .....	11
<b>da</b>	<b>Tommelfingerbandage sport</b> .....	12
<b>fi</b>	<b>Nauhallinen peukalotuki urheilukäyttöön</b> .....	14
<b>sv</b>	<b>Elastiskt band för tumme sport</b> .....	15
<b>el</b>	<b>Αθλητικό strapping αντίχειρα</b> .....	16
<b>cs</b>	<b>Pásková bandáž palce sport</b> .....	17
<b>pl</b>	<b>Bandażowa orteza na kciuk sportowa</b> .....	19
<b>lv</b>	<b>Sporta īkšķa ortoze</b> .....	20
<b>lt</b>	<b>Apjuosiamas nykščio įtvaras sportui</b> .....	21
<b>et</b>	<b>Spordi pöidlarihm</b> .....	22
<b>sl</b>	<b>Opornica za palec šport</b> .....	23
<b>sk</b>	<b>Športová páska na palec</b> .....	25
<b>hu</b>	<b>Hüvelykujj rögzítőpánt sportoláshoz</b> .....	26
<b>bg</b>	<b>Кайшка за палец за спорт</b> .....	27
<b>ro</b>	<b>Chingă pentru degetul mare sport</b> .....	28
<b>hr</b>	<b>Sport traka za palac</b> .....	30

## **SPORT**

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Istrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu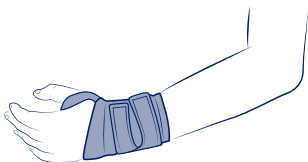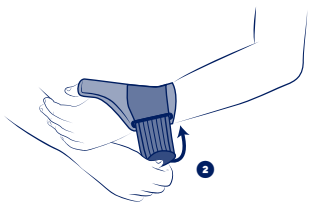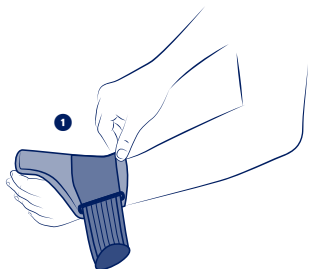

 cm
<b>S</b> 14 - 15,5
<b>M</b> 16 - 17,5
<b>L</b> 18 - 20

<b>en</b>	Wrist circumference
<b>fr</b>	Tour de poignet
<b>de</b>	Handgelenkumfang
<b>nl</b>	Polsomtrek
<b>it</b>	Circonferenza del polso
<b>es</b>	Contorno de la muñeca
<b>pt</b>	Perímetro do punho
<b>da</b>	Håndleddets omkreds
<b>fi</b>	Ranteen ympärysmitta
<b>sv</b>	Handledens omkrets
<b>el</b>	Περίμετρος καρπού
<b>cs</b>	Obvod zápěstí
<b>pl</b>	Obwód nadgarstka
<b>lv</b>	Plaukstas locītavas apkārtmērs
<b>lt</b>	Riešo apimtis
<b>et</b>	Randme übermõõt
<b>sl</b>	Obseg zapestja
<b>sk</b>	Obvod zápästia
<b>hu</b>	Csukló kerülete
<b>bg</b>	Обиколка на китката
<b>ro</b>	Circumferința încheieturii
<b>hr</b>	Obujam ručnog zgloba

en

## **SPORT THUMB STRAPPING**

*Support level 3*

### **Description/Intended use**

This device is a thumb brace providing joint stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition**

Polyamide - polyester - synthetic rubber (SBR) - polyurethane - elastane.

### **Properties/Mode of action**

Thumb stabilisation via a plastic wire stay.

Embossed synthetic rubber (SBR).

A hook-and-loop fastening strap to hold the product in place.

### **Indications**

Thumb osteoarthritis (including rhizarthrosis).

Thumb pain or trauma.

### **Contraindications**

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

Do not use the product if the diagnosis has not been confirmed.

### **Precautions**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

Thread the brace over the wrist and insert the thumb into its housing, ensuring the whole base of the thumb is covered. ❶

Fold the elastic strap over and tighten it around the wrist. ❷

### **Garment care**

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Machine washable at 30 °C. Close the self-fastening tabs before washing. Do not use detergents,

fabric softeners or aggressive products (products containing chlorine, etc.). Do not tumble-dry. Do not iron. Do not dry clean. Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

### **Storage**

Store at room temperature, preferably in the original packaging.

### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2052801 (2024-03)
**THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186